ANNABOX[®]
2go

ENG Instructions for use **ANABOX[®]**
2^{go}

To remove the **ANABOX[®]** 1x7 simply push it out of the bottle holder.

Then the medication dispenser can be filled as follows:

Figure 1: Hold the entire box in your hand.

Figure 2: Now exert light pressure on the cover using the index finger of your other hand.

Figure 3: This makes opening the cover easier with the help of your thumb.

To insert the **ANABOX[®]** 1x7 simply push it back into the holder from the side.

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ENG Tips and information

- Rinse the medication dispenser before first use. To do this please use lukewarm water and a mild cleaning agent, or detergent suitable for cleaning dishes.
- Alternatively you can also use the dishwasher (control temperature 50 °C) and ensure that you do not position the medication dispenser too closely to the heat source.
- We recommend cleaning the medication dispenser before every refill with medication or when a change in medicines occurs.
- It is important to note special storage conditions for medicines such as e.g. light sensitivity, temperature, etc. according to the package insert in the respective packaging.
- If in doubt, please ask your pharmacist or the medication manufacturer.
- We accept no liability for the incorrect use of **ANABOX** or the incorrect intake of medication.
- The medication box should be used by people with mental disabilities only under expert supervision.
- Keep the **ANABOX** out of the reach of children.
- Based on the production date on the back (production times - year/month), the product should be used for 5 years if used properly, if stored and cleaned as instructed.

ENG ANABOX bottle

- Rinse the bottle before first use.
- To do this please use lukewarm water and a mild cleaning agent, or detergent suitable for cleaning dishes. Afterwards let it dry thoroughly.
- When not in use always store the bottle without the lid screwed on.
- The bottle is not intended for use in the microwave.
- Do not pour any hot drinks in it.
- Cleaning in a dishwasher is not recommended.

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anmed GmbH